

TABLE OF CONTENTS

1. Purpose	2
2. Tools & Supplies	2
3. Procedure: Lower Hinge Replacement	2
3.1. Step 1	2
3.2. Step 2	3
3.3. Step 3	3
3.4. Step 4	4
3.5. Step 5	4
3.6. Step 6	5

1. PURPOSE

This document describes the process required to replace the OmniTab Bracket on Nova Carts with a Vesa H block.

2. TOOLS & SUPPLIES

Flat blade screwdriver

T25 Torx security L Key



3. PROCEDURE: LOWER HINGE REPLACEMENT

3.1. STEP 1

Remove channel cover from extrusion near the hinge.



3.2. STEP 2

Remove the screw holding the Upper hinge using the T25 L Key.



3.3. STEP 3

Remove the OmniTab bracket from the arm.



3.4. STEP 4

Removed OmniTab bracket with upper hinge attached.



3.5. STEP 5

Place Vesa H block with the lower hinge on the arm and engage the screw to hold it in place. Tight the screw firmly with the T25 L key until the hinge is flush with the adapter plate.



3.6. STEP 6

Put the channel cover strip removed at Step 1 back to the arm where it came from.



End of Procedure